


8600 Battery Replacement

- 1) Remove 4 Phillips screws on the bottom of the case.
- 2) Remove the back two rubber feet.
- 3) While gently pulling on back cover use a flat blade screw driver to release the plastic catches.

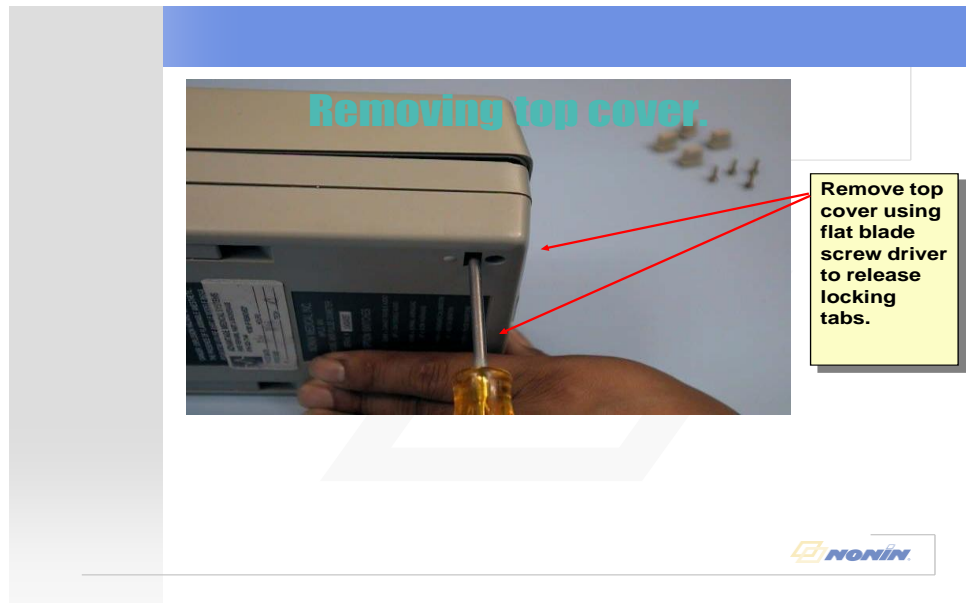

- 4) Unplug the battery.
- 5) Cut the nylon straps that are holding the battery.
- 6) Remove existing battery pack.
- 7) Install new battery make sure battery is in the correct orientation, battery wires closest to the connector.
- 8) Apply the two nylon straps received with the new battery pack.
- 9) Install top cover and watch that the wires in the front of the unit are not pinched when doing so.
- 10) Replace screws and rubber feet.